

Supplements:

1. Please complete your alarm mode selection within 3 minutes after powering on, otherwise you will have to re-start the alarm.
2. If you would like to swap the current alarm mode to a different one, please complete your alarm mode change in 5 seconds, otherwise you will have to re-start the alarm.
3. Use the Music Selection Switch ③ to select the music that you liked. Tune the volume switch ④ to control the volume.
4. For twinkling mode, there are four LED lights on the alarm to show different multi-LED twinkling cycles.
5. If the wetted diaper is not changed in a fairly long time, the alarm will exit the current alarm mode and will automatically replace with only twinkling indicator light ① flashing. If you see the flashing light indication, please change the diaper immediately.

4.Main technical specifications:

1. Operating power: 2 x AAA batteries;
2. Operating Temperature: -30℃--60℃;
3. Operating humidity: 5%--95%.

5. Notes:

1. When the batteries are running low, the power indicator light will flash quickly when you power it on. Please replace with new batteries.
2. To save battery, please turn off the alarm when not in use.
3. To avoid the product damaged by the seep liquid of the batteries, you must take out the batteries when you are not going to use the product for a long time.
4. Do not pull the sensor wire too hard to avoid damage.
5. The batteries are ordinary Carbon batteries and they mainly are for you to test the Product and will last for around a week at best.
6. Please pay attention to the positive and negative polarities of battery.

6.Limited Warranty:

We warrants the product to be free from defects in material and workmanship, arising under normal use, for a period of ONE year from the date of original purchase when brand new. Misuse, loss or improper handling is not covered. We will not pay for transport/insurance costs incurred in returning the product to us authorized reseller, loss of time, inconvenience, loss of use of your product, loss of your product, or any incidental or consequential damages. This warranty extends only to original purchaser and is non-transferable.

We reserves the right to update and alter the products.

Enuresis Alarm



User Manual

**Please read through the instruction manual carefully
before using the alarm**

1. About The Alarm

This SVY001 3-in-1 alarm system is an efficient and inexpensive treatment for nocturnal enuresis, i.e. bedwetting, without using medication. Designed with advanced features, this bedwetting alarm combines MUSIC, VIBRATION or MULTI-LED LIGHT to help even the deepest sleeper wake. The device consists of an alarm unit attached to a sensor via cable connection.

Features:

- Microcomputer control with intelligent design. Multi-alarm modes for you to select easily, including music, vibration and multi-LED light.
- High sensitivity and accuracy.
- Low power consumption.
- Low battery indicator.
- Easy to use, one button operation to set up.

It can be used for:

- Adult bedwetting;
- Baby bedwetting.

2. Package Contents

Name	Quantity	Note
Alarm	1	See FIG.1-A
Manual	1	
Mounting accessories		See FIG.1-B
Sensor + Clamp	1	
Battery	2	

<A> Alarm



 Mounting Accessories




Sensor + Clamp



Battery

FIG.1 PACKAGE CONTENTS

3. Application Instruction:

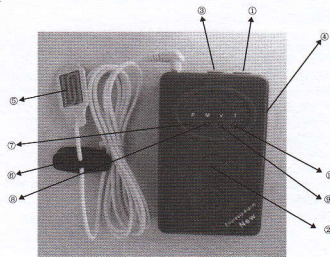

- ①—Power Switch; ②—Alarm mode selection button; ③—Music selection switch;
④—Volume control knob; ⑤—Detector/Sensor; ⑥—Clamp; ⑦—Power indicator light;
⑧—Music indicator light; ⑨—Vibration indicator light; ⑩—Twinkling indicator light.

FIG.2 ILLUSTRATION

How to use (see the FIG.2):

1. Prepare the product.
2. Put the sensor probe ⑤ under or between the diapers, and near to where it will be easily wetted. Then use the clamp ⑥ to clip the diapers to avoid the sensor probe falling off.
3. Push the power switch to "on" position to power on. Wait for about 3 seconds, then press the alarm mode selection button ② for around one second each time to select the alarm mode you like. The first-time press is music mode with music indicator light ⑧. The second-time press is vibration mode with vibration indicator light ⑨. The third-time press is twinkling mode with twinkling indicator light ⑩.
4. When the diaper is wet, the alarm will remind you to change the diaper according to the alarm mode you select.
5. Push the power switch to "off", then change the diaper and wipe the sensor with a clean and absorbent cloth for next time use.
6. Repeat step1 to step5 to set up the alarm mode if you need to use again.